

# Tandem Mobi Insulin Pump

1.

Refreshed just now  
100%  
160 u

**129 →**  
mg/dL

Insulin On Board  
0.5 u

Control-IQ: Active

Bolus

Carbs Glucose

Add Correction Bolus?  
Your glucose is above your Target BG value of 110 mg/dL.

No Yes

Delivery Calculation

Correction 0 u  
Food 0 u  
Insulin On Board 0.5 u  
Total Units to Deliver 0 u

2.

Cancel Bolus Next

0 units

Carbs Glucose

Add Correction Bolus?  
Your glucose is above your Target BG value of 110 mg/dL.

No Yes

Delivery Calculation

Correction 0 u  
Food 0 u  
Insulin On Board 0.5 u  
Total Units to Deliver 0 u

3.

Cancel Bolus Next

0 units

Carbs 0 grams Glucose 129 mg/dL

Delivery Calculation

Correction 0 u  
Food 0 u  
Insulin On Board 0.5 u  
Total Units to Deliver 0 u

4.

Cancel Bolus Next

5.5 units

Carbs 60 grams Glucose 129 mg/dL

Due to IOB, no correction will be calculated. Re-test BG as necessary.

Delivery Calculation

Correction 0 u  
Food 6 u  
Insulin On Board 0.5 u  
Total Units to Deliver 5.5 u

## Tandem Mobi Insulin Pump

5.

Bolus      **Next**

5.5 units

Carbs 60 grams      Glucose 129 mg/dL

**i** Due to IOB, no correction will be calculated. Re-test BG as necessary.

**Delivery Calculation**

Correction	0 u
Food	6 u
Insulin On Board	0.5 u
<b>Total Units to Deliver</b>	<b>5.5 u</b>

6.

**Confirm Bolus**      **Confirm**

5.5 units

Correction N/A

Food 6 u

7.

**Back**      **Confirm Bolus**      **Confirm**

5.5 units


Correction N/A

Food 6 u

Deliver 5.5 u Bolus?      **Cancel**


**Deliver Bolus**

8.

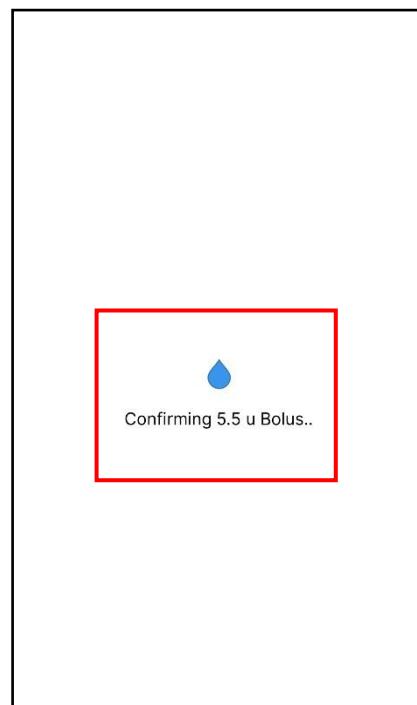


## Tandem Mobi Insulin Pump

9.

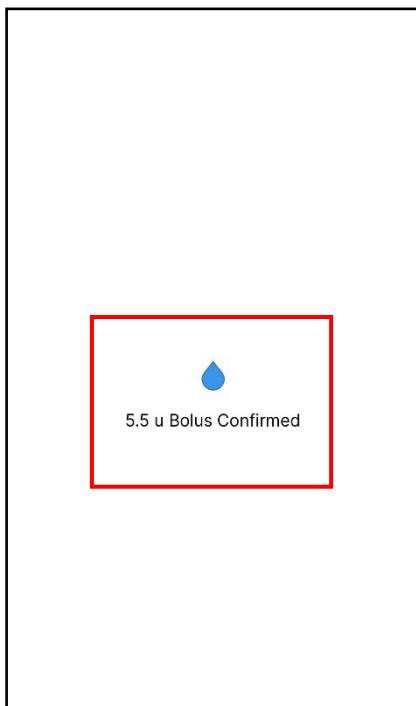

10.

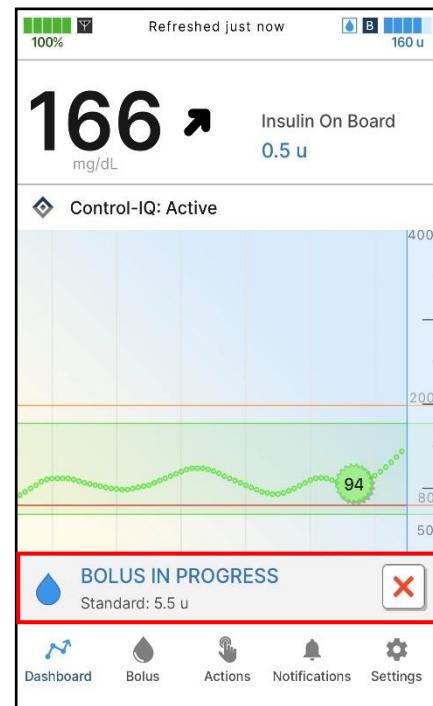

11.

